## **SERIES 2000**

**TECHNICAL DATA SHEET [EN]**

**FICHE TECHNIQUE [FR]**

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## FEATURES

*Series 2000 reusable half masks are designed to be used with twin filters, and to offer the user maximum comfort and durability. With the proper filter, they provide protection against particulates and a wide variety of gases and vapors.*

- Three facepiece options (EPDM, TPE or Silicone).
- Lightweight and with a wide sealing area provides great comfort.
- Low breathing resistance.
- Low profile face that makes it perfect to use with a helmet and/or safety glasses.
- Adjustable head harness for extra comfort.
- Wide field of vision.
- Very intuitive bayonet connection, which ensures a quick and safe filter adjustment.

## IMPORTANT NOTICE TO USERS

*Series 2000* reusable half mask respirators must be used and maintained in accordance with this instruction manual. You are responsible to understand and follow all warnings, restrictions and other information contained in this instruction manual before you enter a contaminated work area. Incorrect use, use of unsuitable spare parts or bad maintenance are dangerous for health and safety and exonerate the manufacturer from all liability. Personal protective equipments for respiratory protection must always be used by specially trained people. The use has to be supervised by a person perfectly aware of the limits of application and of the applicable laws concerning those devices.

### WARNINGS AND RESTRICTIONS TO USER

This half mask does not supply oxygen (O<sub>2</sub>), and must be used in a ventilated work environment with an oxygen volume superior or equal to **17%**.

Use the appropriate filter in each case, depending on concentration and type of contaminant.

- *Series 2000* are twin filter reusable respirators.
- Do not use as protection against Carbon Monoxide.
- Do not alter or modify this respirator.
- Do not use this respirator in containers, pits, sewers or confined places with no ventilation.
- Any facial feature hindering the perfect sealing of this respirator (**e.g. beards**) must be removed.
- If the respiratory equipment is damaged, or if the user has breathing difficulties and/or feels ill or faint, the work area must be abandoned immediately.

### BEFORE EACH USE

Inspect your respirator before and after wearing. Don't wear, and return to your supervisor if:

- Head straps are torn, cut, or have lost elasticity.
- Buckles or head cradle are missing, damaged or broken.
- Facepiece is cracked, torn, distorted, dirty, or has holes.
- Inhale/exhale valves are missing, torn, damaged, or not properly seated.
- Make sure the type of filter selected is appropriate for the specific use of the half mask.
- Filters are cracked, damaged, or not properly seated.
- Any other plastic part is cracked, distorted, or damaged.

### MAINTENANCE

The effective life of the respirator will be influenced by the use conditions and contaminants to which it is exposed. It is highly recommended to follow next steps after every use:

1. Check if the material shows any signs of cracking, wrinkling, or aging. If so, then discard the facepiece immediately.
2. Store the respirator, cleaned, in a sealed bag.

## CERTIFICATION

**MPL Series 2000** are:

- Certified according to Regulation **(EU) 2016/425** relative to Personal Protective Equipment.
- Certified as **Category III PPE**, in accordance with the harmonized standard **EN 140:1998**.
- **CE** marked.

**MPL's** management system is **ISO 9001:2015** certified.

### CLEANING AND DISINFECTION

Cleaning is recommended after each day's use or more frequently if necessary:

1. Remove the filters before proceeding with the cleaning.
2. Gently scrub the facepiece with a soft brush in a mild germicidal detergent, rinse with clean water and air dry. Don't use solvents to wipe or clean the facepiece as these will reduce the life of the respirator and pose a health hazard to the user.
3. Dry the half mask thoroughly.

### REPLACEMENT PARTS

In case of cracking, wrinkling or aging, spare parts can be replaced without using any tools: inhale/exhale valves and head harness assembly.

### STORAGE

When not in use or during storage, do not expose respirator to excessive heat (above **50°C**) or hazardous substances.

Every respirator is supplied in an individual case with this instruction manual for the user included.

MODEL	FACEPIECE	DISPLAY
2000C	EPDM	42 units per box
2000T	TPE	42 units per box
2000S	SILICONE	42 units per box

### LIMITATIONS

Respirators may reduce but do not eliminate wearer exposure to airborne hazards or the risk of contracting any disease or infection. Only use this respirator as part of a comprehensive respiratory protection program. You will receive no respiratory protection if this respirator is not properly fitted and worn. It is imperative that you determine the size and potential hazards that may be present in the environment before selecting appropriate respiratory protection.

## CARTRIDGES & FILTERS

*Series 2000* reusable half mask respirators with must be used with **MPL's** bayonet connection *Series B* cartridges & filters.

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